

INSTRUCTIONS FOR USE

1.

Once a month, after refilling your cat's litter box, spread the entire content of this bag HEALTH INDICATOR® evenly across the top of the litter.

2.

A short moment after your cat uses the litter box, compare the colour of the crystals at the urination spot to the colours on the chart below. The normal colour, when urine is added to the litter, is either light- to dark yellow. Should any other colour become visible, immediately contact your vet (abnormally discoloured litter should be removed).

3.

After completing the test, stir the HEALTH INDICATOR® into the regular litter where it will help eliminate odours.



Homes with multiple cats should separate the cats so each cat has its own litter box with HEALTH INDICATOR®. If your cat typically digs prior to urinating, empty litter box and spread the content of HEALTH INDICATOR® evenly across the litter box. After completing the test, refill the box with regular litter.

NORMAL COLORS



GO VISIT THE VET

